



Apre 2 in 1 Electric Breast Pump

User Manual

- ✓ Memory function memorizes your personal suction cycle
- ✓ Automatic massage function
- ✓ Mains power or battery operated
- ✓ 9 levels of suction pressure and pace
- ✓ Compatible with most major brand bottles
- ✓ Hospital strength ultra quiet motor



- Please read the information on this document carefully and retain for future reference.



Index

1. Features

2. Technical parameters

3. Accessories

4. Assembly method

5. Usage

- Ready to use
- Automatic pumping
- Feeding
- Storing
- Cleaning
- Sterilizing

6. Troubleshooting

7. Attention

8. Packing list

9. Warranty card




1. Features

- Mimic a infant's breastfeeding action
- 15 seconds automatic massage mode
- Great suction, assists with postpartum lactation
- Ultra quiet motor
- 9 levels of suction and pace
- Petal massage cushion can gently and efficiently release breast engorgement
- Remember your last setting
- Both mains and battery operated

2. Technical parameters

- Power: Import 100~240 VAC, 50-60HZ
Export 5V DC, 400mA, wattage <3W
- Battery: AA battery (1.5V*4)
- Pumping intensity: max > 50 Kpa
- Suggested using time: 20-30 mins every time
- Using condition: 5 - 40°C, RH < 80%

3. Accessories:

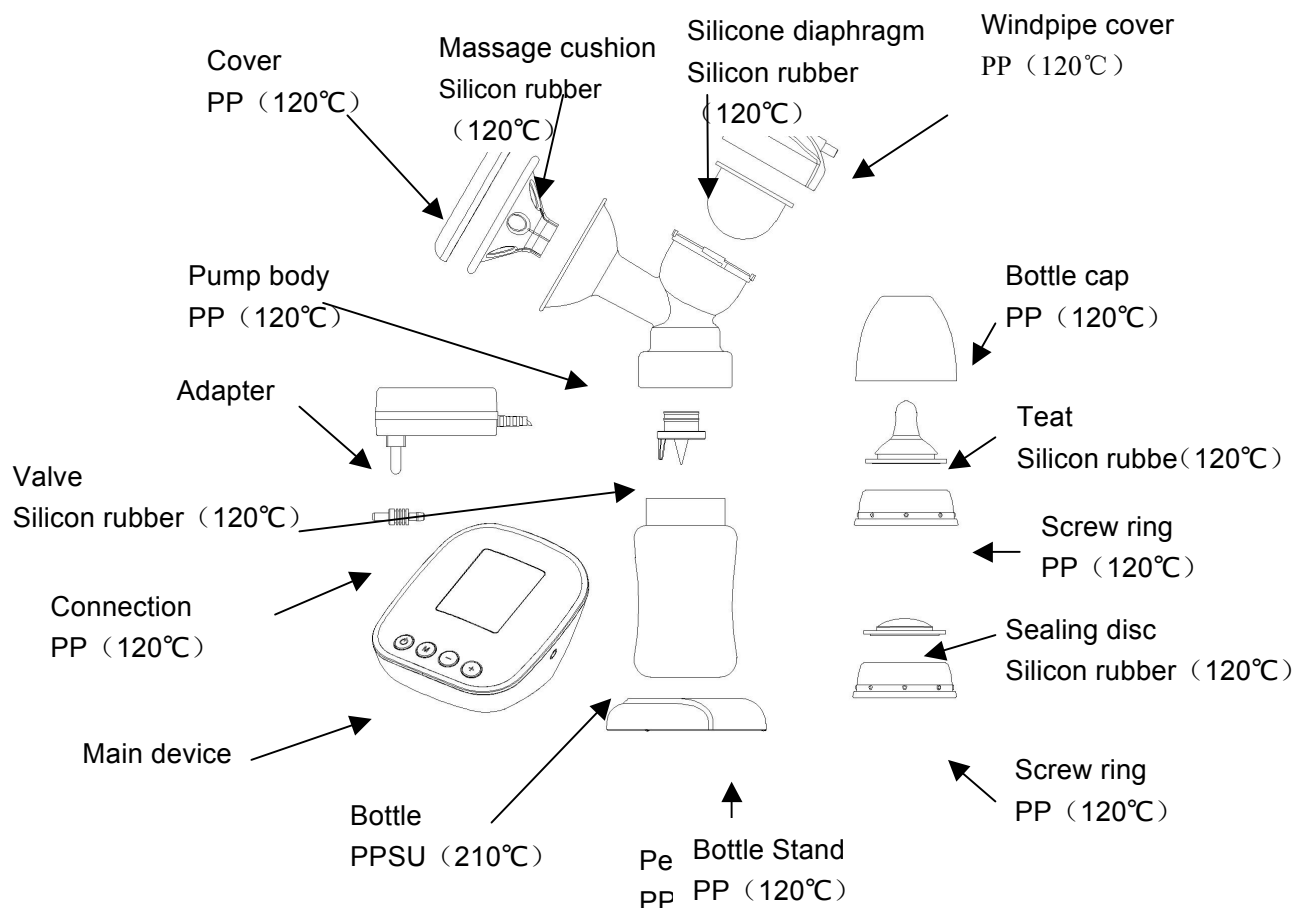



4. Assembly method


Please wash your hands before handling the breast pump.

1. Plug the electric adapter into the electric adapter socket of the main device.


Or

Insert batteries. Ensure that the +and -ends of the batteries are positioned correctly.

2. Please assemble the electric breast pump as below:

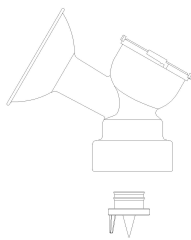


a. Install valve onto lower pump body firmly without any leaks.

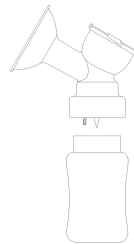


b. Screw bottle onto lower pump body.




c. Push silicone diaphragm onto the lower pump body firmly without any leaks.

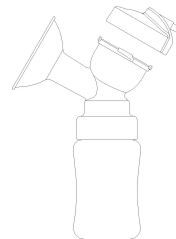


d. Connect air tube onto upper pump body and the air tube socket of the main device.

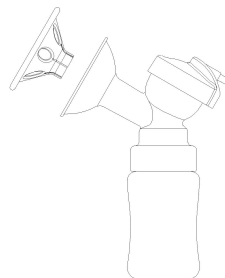

e. Make sure the massage cushion is firmly attached without any leaks. Use your thumb to press the massage cushion, then push the massage cushion into the pump body as deep as possible.

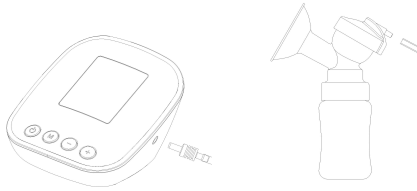

g. Screw the connection into the pump body.

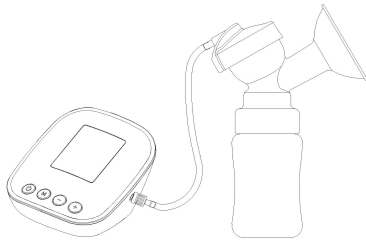

Please cover the massage cushion and put the bottle on the pedestal when not use.

5. Usage:

1. Ready to use- TIPS:

After 1~4 days of childbirth, you may use the electric breast pump.

- You can listen to some soft relaxing music which will help your lactation.
- Use a cleaning cotton ball to wipe the nipples and areolas.
- Use a heat towel to warm your breasts.
- Massage your breasts with fingers to stimulate lactation and make you comfortable.

2. Start to use

- Adjust your position and make sure that your are comfortable.
- Grip the pump body and aim the massage cushion to the breast.
- Turn on the Power button to start function.

3. Massage mode

- After switching on the device, it will automatically starts with 15 seconds of “massage mode” and then “pump mode”.
- When you press the “+” or “-”, breast pump will change “massage mode” to “pump mode”.
- When you press the “M” button again, the breast pump will back to the “massage mode”

4. Pump Mode

- After 15 seconds of “massage mode” you will have “pump mode” afterwards.
- Please relax which will help you lactate.
- Change to the other breast when complete.



5. Adjust the suction and the pace

- Press the +/- to adjust the level.
- **We recommend you to start pumping the milk with level 1 of suction and pace for your first use.**
- After you are familiar with the operation, you can adjust the level depending on your situation and needs.
- When you need to release from milk engorgement, you can try a higher level of suction.
- You can press the “M” button for longer than 2 seconds to switch between “massage mode” and “pump mode”.
- You can change to “massage mode” after pumping to release your breast or stimulate lactation.

7. Switching off

- When finish pumping, press the POWER button and unplug the adapter.

8. Cleaning

- Please clean the pump body, massage cushion, valve, bottle, teat, screw ring, sealing disc immediately after every use.
- Cover, sealing ring, pedestal, and connection need cleaning every week.
- **Attention: Clean the valve carefully not to break the tip.**
Never use a brush or detergent to clean.

9. Sterilizing

- Please clean and sterilize all PP and silicone accessories. TIPS BELOW:

a. Scalding
<ul style="list-style-type: none">• Put the accessories in the boiling water for 5 mins• Never let the accessories touch the side of the pots to prevent from distortion
b. Steam sterilization by the microwave
<ul style="list-style-type: none">• Please use sterilization containers for microwave sterilization• Please sterilize in the microwave according to the information.
c. Steam sterilization
<ul style="list-style-type: none">• All the accessories can be sterilized by a steam sterilizer.
D. Other methods
<ul style="list-style-type: none">• Never use dry heat, UV or dish-washing machine to sterilize.

Attention: Please keep all parts dry after sterilizing

Please follow sterilizer instructions.

10. Storing milk

- Store the milk in either the fridge or freezer after collecting.



Storing time for freezing

	Freezing chamber	Refreshing chamber
Fresh milk	3 months	24 hours
Defrosted milk	--	24 hours

Attention: Milk can't be frozen again after defrosted.

Never defrost or warm the milk by microwave.

Please do not store the milk at room temperature.

11. Feeding baby

- Please check the temperature before feeding.
- Take out the bottle from the pump body and fit on the teat.
- Feed baby when the milk is in suitable temperature.
- Frozen milk needs to defrost and warm up to 38-42°C before feeding the baby.



6. Troubleshooting

If facing the following troubles during pumping, please consult the table below:

Trouble	Cause	Troubleshooting
Weak suction	1. Connection part is leaking 2. Tube is blocked 3. Ensure the massage cushion fits on the breast	Check the connection Don't bend the tube Clean and adjust
No suction	1. Have no valve or the valve has been broken 2. The valve or tube has been broken during sterilizing	Install the valve Change the accessories
Can't switch on	1. Wrong adapter 2. Check batteries	Change the adapter Change the batteries
Milk back flow	1. Using a stronger suction level than necessary 2. There is water in the tubes 3. Bottle is too small	Choose the a lower level Dry or change the tubes Change the bottle to another brand

7. Attention:

- The product is individual, do not share with other people.
- Always unplug when it is not use.
- Never use the breast pump in a damp place or in water.
- If the main device drops into the water, do not touch it and please unplug with dry tools.
- Please keep children out of reach.
- All the accessories can't be over boiled
- Don't use the pump in excess of 20~30 mins on each side.
- If you have any discomfort, stop using it immediately and consult a doctor.
- For adult use only.
- Our company retains the right of design alteration, and may not be able to inform directly.



8. Packing list

No.	Name	Material	Quantity	No	Name	Material	Quantity
1	Massage cushion	PP	1PCS	10	Sealing ring	PP	1PCS
2	Cover	Silicon rubber	1PCS	11	Teat	Silicon rubber	1PCS
3	Pump body	PP	1PCS	12	Silicone diaphragm	Silicon rubber	1PCS
4	Screw ring	Silicon rubber	1PCS	13	Bottle cap	PP	1PCS
5	Windpipe cover	PP	1PCS	14	Sealing disc	Silicon rubber	1PCS
6	Windpipe	Silicon rubber	1PCS	15	Valve	PP	1PCS
7	Connection	PP	1PCS	16	Main device		1PCS
8	Bottle	PPSU	1PCS	17	Adapter		1PCS
9	Pedestal	PP	1PCS	18			



9. Warranty card:

Thank you for choosing, please keep the warranty below or your receipt.

If have any question, please consult us at www.aprebaby.com.au

Name		Tel	
Product name		Mode	
Purchase shop		Purchase date	
Address			
Service			

Attention

- **Apre provides one year warranty service for main device and are not responsible for improper use.**